

# Product Specification



## ORALTOX Saliva Drug Tester



CKO-ALCO-DRUG



Locking Handle

Indicator Window

Saturation Indicator Strip

Sponge

### PRODUCT OVERVIEW

This drug screening solution detects a multitude of drug types through a discrete, convenient, and non-invasive single use collection swab.

Perfect for professional use, the device can quickly measure an employee's readiness to work. After swabbing the inside of the user's mouth, and colour appears on the saturation indicator strip, a negative result will appear within 2 minutes (10 minutes for an inconclusive result). This product is a hygienic, low cost, disposable option.

### RESULTS INFORMATION

<b>ALC</b>	Alcohol > .02% BAC
<b>KET</b>	Ketamine 50 ng/mL
<b>AMP</b>	Amphetamine 50 ng/mL
<b>MET</b>	Methamphetamine 50 ng/mL
<b>BAR</b>	Barbiturates 50 ng/mL
<b>MTD</b>	Methadone 30 ng/mL
<b>BUP</b>	Buprenorphine 5 ng/mL
<b>OPI</b>	Opiates 40 ng/mL
<b>BZO</b>	Benzodiazepines 10 ng/mL
<b>OXY</b>	Oxycodone 20 ng/mL
<b>COC</b>	Cocaine 20 ng/mL
<b>PCP</b>	Phencyclidine 10 ng/mL
<b>COT</b>	Cotinine 50 ng/mL
<b>PPX</b>	Propoxyphene 50 ng/mL
<b>EDDP</b>	Methadone metabolite 20 ng/mL
<b>THC</b>	Marijuana 50 ng/mL (parent compound)

**FOR MORE INFORMATION, CONTACT OUR SALES TEAM.**

**T:** 020 8863 8333 (OPT1) • **E:** sales@c-ko.co.uk • **W:** www.c-ko.co.uk

# Product Specification



## ORALTOX Saliva Drug Tester


### Ease of Use

This single-use drug screening product is easy to use, suitable for all environments. No training required, simply follow the included instructions step by step.

#### Step 1:

Firstly, ensure that the user does not have anything in their mouth (such as food, drink, gum) 10 minutes prior to collection. Instruct the user to actively swab the inside the mouth, top of tongue, and between the cheek and the gum.

Hold the swab in the users closed mouth until either colour appears on the saturation indicator strip, or until 4 minutes has passed.

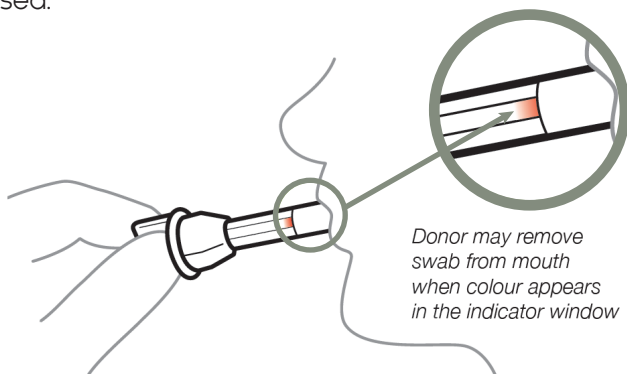


#### Step 2:

Position the test device upright on a flat surface and insert the collection swab sponge first. Push and turn the swab until the threaded handle locks into the top of the device.

#### Step 3:

The test will reveal either negative, inconclusive (non-negative) or invalid results. Negative results can be read at 2 minutes, inconclusive results can be read at 10 minutes. Do not read test results after 15 minutes. After results are interpreted, discard device in regular trash receptacle.

### RESULTS INFORMATION

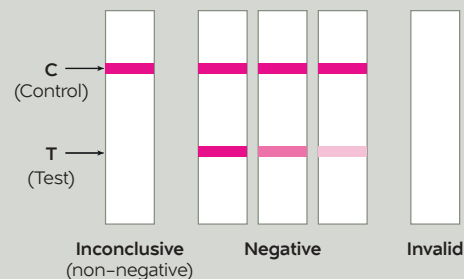

#### Test Inconclusive

An 'Inconclusive' result is indicated by a reddish-purple 'Control' line at the top of the test, with no second 'Test' line appearing at the bottom to indicate drug usage (as per the listed quantities ng/mL).

#### Test Negative

A 'Negative' result is indicated by a reddish-purple 'Control' line and a second reddish-purple 'Test' line appearing beneath to indicate the use of any listed drug quantities (ng/mL).

The intensity of the test lines may vary. Regardless of intensity, colour, or size, a line indicates a negative result for that drug.

#### Test Invalid

If a reddish-purple 'Control' line fails to form at the top of the test, then the test results are categorised as invalid and must not be used.



**FOR MORE INFORMATION, CONTACT OUR SALES TEAM.**

**T:** 020 8863 8333 (OPT1) • **E:** sales@c-ko.co.uk • **W:** www.c-ko.co.uk