

Product Specification



DRIVESAFE Express Single Use



CKO-ALCO-SINGLE

PRODUCT OVERVIEW

The DRIVESAFE Express Single Use is a discrete, convenient, and non-invasive single use breath alcohol tester that provides reliable results with easy-to-read colour changing crystals. It is a hygienic, low cost, disposable option.

Perfect for professional use, the device can quickly measure an employee's readiness to work. When the user blows through the tube, the crystals change colour upon detection of alcohol. The more intense the colour, the higher the alcohol concentration. The four-level indicator accurately identifies your BrAC within 2 to 4 minutes.

RESULTS INFORMATION

Clear breath alcohol concentration (BRaC) results with 2 to 4 minutes, indicated by the colour changing crystal (please see the below table).

| | 0,0 ‰ | 0,2 ‰ | 0,5 ‰ | 0,8 ‰ |
|------------------------|-------|-------|-------|-------|
| Canada (mg/dL) | 00 | 20 | 50 | 80 |
| Europe (mg/L) | 0.00 | 0.10 | 0.25 | 0.35 |
| Scandinavia (‰) | 0,0 | 0,2 | 0,5 | 0,8 |
| USA (%) | .00 | .02 | .05 | .08 |

FOR MORE INFORMATION, CONTACT OUR SALES TEAM.

T: 020 8863 8333 (OPT1) • **E:** sales@c-ko.co.uk • **W:** www.c-ko.co.uk

Product Specification



DRIVESAFE Express Single Use

Ease of Use

This single-use breathalyzer is an easy to use solution, suitable for all environments. No training required, simply follow the included instructions step by step.


Step 1:

Firmly push both ends of the tube to break the security foil.




Step 2:

Blow for 10 seconds in the direction of the arrow.

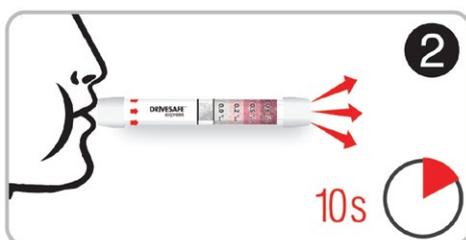

Step 3:

Wait 2 to 4 minutes and compare the colour of crystals to the printed scale to estimate your BrAC level.

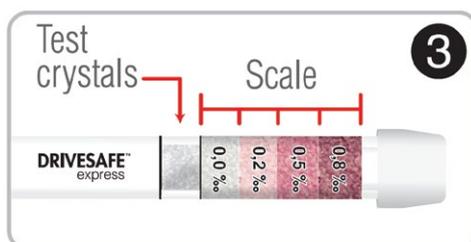

Step 4:

If you are over the limit, call a friend or cab for a safe ride home.



FOR MORE INFORMATION, CONTACT OUR SALES TEAM.

T: 020 8863 8333 (OPT1) • **E:** sales@c-ko.co.uk • **W:** www.c-ko.co.uk